RS VIVO

RS VIVO Guide

INSTRUCTIONS FOR USE

rapidshape

PACKAGING UNITS

RS VIVO Guide, 1000g [RS006196]

DESCRIPTION

RS VIVO Guide is a light-curing resin for the additive production of dental surgical guides. The material is optimized for wavelengths of 385 in layer thicknesses of 100 µm and can be used with 3D printers of following manufacturers:

- Rapid Shape [D-Series, PRO-Series, ONE]

COMPOSITION

Preparation made of acrylates and initiators.

INDICATION

Additive manufacturing of dental surgical guides.

CONTRAINDICATIONS

Patient and user: Products made of RS VIVO Guide must not be used in patients or by users who are known to be allergic to any of the ingredients. Otherwise, the patient group is not restricted.

ADVERSE REACTIONS

Patient and user: Product may cause allergic reaction. User: Observe warnings.

INTERACTIONS

Interactions are not known.

USE

BUILD PROCESS

- **Prepare data:** The orientation and type of support structures can be found in the instructions for use provided by the system supplier.
- **Select variable process parameters:** The selection of suitable process parameters is carried out according to the specifications of the printing system supplier. Make sure to select the correct material-specific parameter set (product, layer thickness, exposure time).
- **Pour the material into the material reservoir of the printer.** Let the material rest for 10 min in the material reservoir.
- **Please note that the „Validated Surgical Guide Build Plate“ must be used for the printing of this material.**
- **Start build process.**

CLEANING AND DRYING

- **Clean part:** The object (with support structure) must be still mounted on the Surgical Guide Platform. The part is cleaned:
 - Option 1:** Under flowing isopropanol for max. 10 min cleaning
 - Option 2:** Clean the part with RS Wash / PROWASH (Rapidshape) devices is conducted by selecting the

corresponding materials parameter set (300s pre-cleaning / 300s cleaning / 90s drying). If necessary, carefully clean the objects with a brush and a spray bottle filled with isopropanol.

- **Dry component:** Blow off component with compressed air and allow to dry for 30 minutes. Alternatively, the objects can be stored in a drying cabinet at a temperature of approx. 37°C. The object must be still mounted on the Surgical Guide Platform.

POST-CURING

- For post-exposure, the components must be dry and cooled down to room temperature. The object must be still mounted on the Surgical Guide Platform.
 - **Option 1:** Post-curing in the light unit RS Cure / PROCURE (Rapidshape) with selection of the corresponding parameter set of the material. [Exposure time: 6 min / vacuum 50 mbar / without nitrogen / heater off / power 75 % / wavelength 375+415 nm].
 - **Option 2:** The post-exposure is performed in the Otoflash G171 (or unit identical in construction) post-curing unit from NK-Optik with 3000 flashes each for the top and bottom of the object (Σ 6000).
Recommendation: Curing under exclusion of oxygen prevents the formation of the inhibition layer.

The curing device must be maintained and calibrated according to the manufacturer's specifications to ensure mechanical properties, color and biocompatibility.

- **Separate part from the platform and remove the support structure.**

FINAL CLEANING (Optional)

- **Option 1:** Clean the object under flowing isopropanol for max. 2 min. If necessary, carefully clean the objects with a brush and a spray bottle filled with isopropanol.
- **Option 2:** Cleaning the part with RS Wash / PROWASH (Rapidshape) devices is conducted by selecting the corresponding material parameter set for "Final Cleaning" (0s pre-cleaning / 40s cleaning / 90s drying). Afterwards blow off the component with compressed air and allow to dry.

➤ POST-PROCESSING COMPLETED

FURTHER PROCESSING INSTRUCTIONS

Parts made of RS VIVO Guide are finished according to the standard dental technician methods.

INFORMATION ON OPTIONAL STERILISATION

RS VIVO Guide is sterilisable with moist heat (autoclave with fractionated vacuum) under following conditions:

- **US:** 132°C (270°F) for 4 min. / Drying for 30 min.
- **Outside US:** 132°C (270°F) for at least 3 min / Drying according to local practice.

WARNINGS

Any deviation from the processing process described here can jeopardize the biocompatibility, functionality and safety of the final product. Contains skin sensitizing substances (acrylates, phosphinate). Please always observe the respective valid safety data sheets. Applies to the uncured material: May cause an allergic skin reaction. Avoid contact with unpolymerized material. Wear protective gloves / protective clothing / eye protection. If on skin: Wash with plenty of soap and water. Avoid breathing dust / vapours / spray. Switch on extraction system. If skin irritation or rash occurs: Get medical advice / attention. The unpolymerized material is harmful to the environment. Avoid release to the environment. Collect spillage. Always observe

the relevant country-specific health and safety regulations and legal requirements when handling hazardous substances.

STORAGE INSTRUCTIONS / SHELF LIFE

Protect from exposure to light. May polymerize spontaneously. Store between 4°C and 25°C (39°F-77°F). Close after use. Do not use after the expiration date. See imprint/label for expiration date.

DISPOSAL INSTRUCTIONS

Dispose of contents/container in accordance with local regulations.

IMPORTANT PRODUCT INFORMATION

R_x Federal (U.S.) law restricts the finished device to sale by or on the order of a dentist.

The product was developed for use in dentistry and must be used according to the instructions for use. Serious incidents must be reported to DeltaMed GmbH and the responsible authority.

	Batch code
	Catalogue number
	Medical device
	Use-by date
	Consult instructions for use
	Temperature limit
	Keep away from sunlight
	Stir before use
	Manufacturer
	Manufacturing Date
	Distributor
	Exclamation mark
	Environmental hazard

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**Rapid Shape GmbH**