**rs
VIVO**

RS VIVO Denture Flex

Transparent
Pink-Transparent

rapidshape

bg	Инструкция за употреба	4
cs	Návod k použití	7
da	Brugsanvisning	10
de	Gebrauchsanweisung	13
el	Οδηγίες χρήσης	16
en	Instructions for use	19
en [USA]	Instructions for use	22
es	Modo de empleo	28
et	Kasutusjuhend	31
fi	Käyttöohje	34
fr	Mode d'emploi	37
hr	Upute za uporabu	40
hu	Használati útmutató	43
it	Istruzioni per l'uso	46
lt	Naudojimo instrukcija	49
lv	Lietošanas instrukcija	52
nl	Gebruiksaanwijzing	55
no	Brugsanvisning	58
pl	Instrukcja stosowania	61
pt	Manual de instruções	64
ro	Instrucțiuni de utilizare	67
ru	Инструкция по применению	70
sk	Návod na použitie	73
sl	Navodila za uporabo	76
sv	Brugsanvisning	79
tr	Kullanım kılavuzu	82

ORDERING INFORMATION

RS VIVO Denture Flex, 1000 g

Part no.

Transparent

04718 [Rapid Shape part no.: RS006207]

Pink-Transparent

04719 [Rapid Shape part no.: RS006218]

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Flexible denture bases

Patient target group

Persons being treated in the context of a dental procedure.

Intended users

Dentist, dental technician

Suitable for the following DLP printers/cleaning/post-exposure

see "Annex 1" (enclosed separately)

Processing

- The properties of the final product depend, among other things, on post-processing. Correct post-exposure is important for biocompatibility. Therefore it must be ensured that the light unit is in an orderly condition and that the moulds are completely cured (refer to the process description).
- Before use, the material in the bottle should be shaken intensively and homogenized with a bottle roller.
- Polish surface mechanically.
- Processing temperature $23\text{ }^{\circ}\text{C} \pm 2\text{ }^{\circ}\text{C}$.

Safety information

- Only for the specified intended use by trained specialists.
- Avoid direct contact with the liquid material and the components before post-curing, in particular in pregnant/breastfeeding women. Irritating to eyes and skin (sensitization possible).
- Wear personal protective equipment (protective gloves, goggles) when handling the uncured material.
- Wear suitable personal protective equipment (protective gloves, goggles, face mask) when finishing the cured material.
- After contact with eyes rinse thoroughly with water immediately and consult a doctor.
- After contact with skin wash immediately with water and soap.
- Biocompatibility is only guaranteed with complete polymerisation.
- MMA free.
- Refer to the relevant safety data sheet for hazard and safety information.

Cleaning

By dental laboratory / practice

In dental laboratories and practices, the cleaning is done in ultrasonic baths.

By the patient

For daily cleaning of the prosthesis, use lukewarm water, a soft toothbrush, liquid soap or detergent. Then rinse well afterwards. Denture cleaning tablets can be used following the manufacturer's instructions.

Note

- Do not use toothpaste; the abrasive particles are coarse and scratch the denture.
- Do not use any acids or solvents.

Notes

- Detax shall not be held liable for any damage caused by misuse.
- Always keep container tightly sealed, immediately close the container carefully after each use.
- Read and understand the safety data sheet!
- For users and/or patients

Any serious incidents occurring in relation to this product should be reported immediately to incident@detax.com and to the competent authority of the Member State in which the user and/or patient is established.

Storage

- RS VIVO Denture Flex is to be stored dry (at 15 °C - 28 °C) and protected from light. Minimal influence of light can already induce polymerisation.
- To protect against contamination, cover the material in the tray with the lid or a glass plate.

Contraindication

Contains (meth)acrylics and phosphine oxide. Some ingredients of RS VIVO Denture Flex may cause allergic reactions in predisposed persons. In such cases refrain from using the product. RS VIVO Denture Flex only insert intraorally in completely polymerised state.

Adverse effects

Product may cause allergic reactions.

Disposal

Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

Storage

15 °C
59 °F

28 °C
82 °F

Processing
at 23 °C ± 2 °C

Manufacturing

Data preparation and fabrication of the support structure according to the instructions of the CAD software manufacturer.

① Construction process

Generation of a Print Job complying with machine and material parameters.

② Post-processing

After raising the platform, a drip time of approx. 10 minutes is recommended. If possible, post-processing should commence immediately following the construction process.

③ Cleaning

see “Annex 1, Cleaning equipment”

④ Post-exposure

see “Annex 1, Light curing equipment”

⑤ Surface processing

Polish surface mechanically.

Caution statement

Caution: Federal Law restricts this device to sale by or on the order of a dentist.

Indications for use

The RS VIVO Denture Flex is a light-curable polymerizable resin intended to be used in conjunction with extra-oral curing light equipment. The RS VIVO Denture Flex is indicated for the fabrication, by additive manufacturing, of dental objects such flexible partial removable dentures.

Patient target group

Persons being treated in the context of a dental procedure.

Intended users

Dentist, dental technician

Special manufacturing requirements

The printing machine and post-curing unit should be set-up, validated and maintained according to their labelling and instructions for use. RS VIVO Denture Flex is only intended to be used with the identified compatible equipment in Annex 1.

Environmental conditions

Processing temperature $23^{\circ}\text{C} \pm 2^{\circ}\text{C}$ / $73^{\circ}\text{F} \pm 4^{\circ}\text{F}$.

Cleaning supplies

Rinse bathtubs, flush cutter, paper towel, squeeze bottle for isopropyl alcohol, scraper

Processing

- The use of RS VIVO Denture Flex is only approved when applied with the compatible devices mentioned in this instruction for use as outlined in Annex 1.
- Generate the object (STL-file) using a commercial CAD software, which is intended for dental applications.
- The properties of the final product depend, among other things, on post-processing. Correct post-exposure is important for biocompatibility. Therefore it must be ensured that the exposure device is in an orderly condition and that the moulds are completely cured (observe manufacturing process).
- After storage, the material in the bottle should be shaken and homogenized with a bottle roller before use.
- Do not use heat-based methods for disinfection or sterilisation. This could possibly deform the workpiece.

Technical Data

ISO 20795-1-Type 4

Does not contain MMA monomer.

Does not contain phthalates.

Processing

at $23^{\circ}\text{C} \pm 2^{\circ}\text{C}$ / $73^{\circ}\text{F} \pm 4^{\circ}\text{F}$

Storage

15°C
 59°F

28°C
 82°F

Safety information

- Only for the specified intended use by trained specialists.
- Avoid direct contact with the liquid material and the components before post-curing, especially in pregnant/breastfeeding women. Irritating to eyes and skin (sensitization possible).
- Wear personal protective equipment (protective gloves, goggles) when handling the uncured material.
- Wear suitable personal protective equipment (protective gloves, goggles, face mask) when finishing the cured material.
- If resin gets into your eyes immediately rinse and consult a doctor.
- After contact with skin wash immediately with water and soap.
- Biocompatibility is only guaranteed with complete polymerisation.
- Refer to the relevant safety data sheet for hazard and safety information.

Notes

- Detax shall not be held liable for any damage caused by misuse.
- Always keep container tightly sealed, immediately close the container carefully after each use.
- Read and understand the safety data sheet!

MRI Safety Information

RS VIVO Denture Flex is MR Safe as it is composed of materials that are electrically nonconductive, nonmetallic and nonmagnetic.

Storage

RS VIVO Denture Flex is to be stored dry at 15 °C - 28 °C / 59 °F - 82 °F and protected from light. Minimal influence of light can already induce polymerisation. Please cover the material vat with its lid or a glass plate to protect material from contamination.

Contraindications

Contains (meth)acrylics

Some ingredients of RS VIVO Denture Flex may cause allergic reactions in predisposed persons. In such cases refrain from using the product. RS VIVO Denture Flex only insert intraorally in completely polymerised state.

Adverse effects

Product may cause allergic reactions.

Disposal

Dispose any remaining resin material from the printer after use. Do not reuse leftover resin. Disposal of the contents/container must be carried out in accordance with the local/regional/national and international regulations.

Material properties RS VIVO Denture Flex

Color*	transparent, pink-transparent
Viscosity*	1000-1700 mPas
Flexural strength**, ***	≥ 25 MPa
Bending modulus**, ***	≥ 900 MPa
Water sorption**	$< 32 \mu\text{g}/\text{mm}^3$
Water solubility**	$< 3.5 \mu\text{g}/\text{mm}^3$

*applies to liquid resin; **applies to cured objects; ***according to internal design and requirements specifications

Manufacturing process

① Mixing

Before filling resin into the reservoir, the material must be mixed. Use a bottle roller or mix/shake it by hand. Inadequate mixing could cause deviations of color and print failures.

② Filling of printer

Fill 3D printing resin in the reservoir of the printer. It is not allowed to pour 3D printing resin from the reservoir of the printer back into the resin bottle.

③ Construction process

After storage, the material in the bottle should be shaken intensively and homogenized with a bottle roller before use. Generate a print job complying with printer and material parameters. For setting up the printer follow the instructions for use of the printer. Data preparation and fabrication of the support structure according to the instructions of the CAD software manufacturer. Remaining print resin material after printing is not allowed to be poured back into the resin bottle.

④ Post-processing

If possible, post-processing should commence immediately following with this construction process. After raising the platform, a drip time of approx. 10 minutes is recommended. Remove the platform from the printer and remove the construction components using appropriate device (e.g., knife or spatula). Carefully remove excess resin using a light flow of compressed air.

⑤ Cleaning

The construction components should be cleaned in two steps with isopropyl alcohol (purity $\geq 98\%$) using an ultrasonic bath. For additional suitable cleaning unit refer to Annex 1, item "Cleaning equipment".

1. Clean the construction components for 3 minutes in a reusable isopropyl alcohol (purity $\geq 98\%$) using an ultrasonic bath. Then carefully remove the construction components from the support structure.
2. The precleaned construction components must be cleaned thoroughly for 3 minutes using a fresh isopropyl alcohol (purity $\geq 98\%$) with an ultrasonic bath. Prior to post-exposure, check the construction components for residues. Then blow off with compressed air. After cleaning the surface of the construction components must no longer be sticky (dry) and not shiny (matt).

⑥ Post-exposure

Post-exposure is performed with a suitable curing unit (See Annex 1, item „Light curing equipment“).

Caution

The use of RS VIVO Denture Flex is only approved when applied with the compatible devices mentioned in this instruction for use. Any unauthorized changes to the process equipment, parameters, or software may result in a device that is out of specification.

Finishing

Finish the construction components by using conventional dental methods and instruments.

Symbol explanation

Graphic	Reference number	Title	Description	Reference
	5.1.1	Manufacturer	Indicates the medical device manufacturer.	ISO 15223-1:2021 Medical devices - Symbols to be used with information to be supplied by the manufacturer - Part 1: General requirements
	5.1.4	Use-by date	Indicates the date after which the medical device is not to be used.	ISO 15223-1:2021 Medical devices - Symbols to be used with information to be supplied by the manufacturer - Part 1: General requirements
	5.1.5	Batch code	Indicates the manufacturer's batch code so that the batch or lot can be identified.	ISO 15223-1:2021 Medical devices - Symbols to be used with information to be supplied by the manufacturer - Part 1: General requirements
	5.1.6	Catalogue number	Indicates the manufacturer's catalogue number so that the medical device can be identified.	ISO 15223-1:2021 Medical devices - Symbols to be used with information to be supplied by the manufacturer - Part 1: General requirements
	5.1.9	Distributor	Indicates the entity distributing the medical device into the locale.	ISO 15223-1:2021 Medical devices - Symbols to be used with information to be supplied by the manufacturer - Part 1: General requirements
	5.3.2	Keep away from sunlight	Indicates a medical device that needs protection from light sources.	ISO 15223-1:2021 Medical devices - Symbols to be used with information to be supplied by the manufacturer - Part 1: General requirements
	5.3.7	Temperature limit	Indicates the temperature limits to which the medical device can be safely exposed.	ISO 15223-1:2021 Medical devices - Symbols to be used with information to be supplied by the manufacturer - Part 1: General requirements
	5.4.3	Consult instructions for use or consult electronic instructions for use	Indicates the need for the user to consult the instructions for use.	ISO 15223-1:2021 Medical devices - Symbols to be used with information to be supplied by the manufacturer - Part 1: General requirements

Graphic	Title	Description	Reference
	Prescription only	Requires a prescription in the United States of America Caution: In the United States of America, federal law restricts this device to sale or use by, or on the order of, a dentist.	USA Code of Federal Regulations 21 CFR Part 801 § 801.109 (b)(1)
	Magnetic resonance	The RS VIVO Denture Impact is MR Safe.	FDA Guidance document: Testing and Labeling Medical Devices for Safety in the Magnetic Resonance (MR) Environment dated May 20, 2021.
	Warning	The uncured material is considered as hazardous substance.	29 CFR 1910.1200 Occupational Safety and Health Standards
	This way up	This is the correct upright position of the distribution packages for transport and/or storage.	ISO 780:2015(E) Packaging-Distribution packaging- Graphical symbols for handling and storage of packages

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detax GmbH
Rapid Shape GmbH