

ANNEX 1

Digital light Processing (DLP) printer, operation software and parameter

Printer manufacturer	Printer type	Light source	Light intensity	Operation software	Print parameter data set*
rapidshape	ONE	385 nm	3.5 mW / cm ²	Rapid Shape Print Studio	RS VIVO Denture Impact
rapidshape	PRO 20	385 nm	2.0 mW / cm ²	Rapid Shape Print Studio	RS VIVO Denture Impact
rapidshape	D20 Series	385 nm	2.0 mW / cm ²	Netfabb 2020	RS VIVO Denture Impact
rapidshape	PRO 30	385 nm	2.0 mW / cm ²	Rapid Shape Print Studio	RS VIVO Denture Impact
rapidshape	D30 Series	385 nm	2.0 mW / cm ²	Netfabb 2020	RS VIVO Denture Impact
rapidshape	D40 Series	385 nm	2.0 mW / cm ²	Netfabb 2020	RS VIVO Denture Impact
rapidshape	D50 +	385 nm	2.0 mW / cm ²	Netfabb 2020	RS VIVO Denture Impact

* The set of parameters includes all relevant material- and printer specific information.

Cleaning equipment

Device manufacturer	Device type	Cleaning process
rapidshape	RS wash, PRO wash, WASH	<p>Use the following settings: rapidshape RS VIVO Denture Impact</p> <p>Prior to post-exposure, check the openings, cavities and gap areas for residues. Then blow off with compressed air.</p>
Bandelin Sonorex	Ultrasonic bath	<p>Clean the parts with isopropyl alcohol (purity $\geq 98\%$) for 3 minutes. Then thoroughly clean the openings, cavities and gap areas with compressed air.</p> <p>The main cleaning is performed in a separate vessel with fresh isopropyl alcohol (purity $\geq 98\%$) for 3 minutes.</p> <p>Prior to post-exposure, check the openings, cavities and gap areas for residues. Then blow off with compressed air.</p>

Light curing equipment

Device manufacturer	Device type	Curing process
rapidshape	RS cure, PRO cure, CURE	<p>Use the following settings: rapidshape RS VIVO Denture Impact</p>
NK Optik	Otoflash G171	<p>2 x 2000 flashes under inert gas, turn around components after 2000 flashes</p>
NK Optik	Otoflash 250/500	<p>4000 flashes under inert gas @15 Hz</p>



detax GmbH
Carl-Zeiss-Str. 4
76275 Ettlingen/Germany
www.detax.com



Rapid Shape GmbH
Roemerstr. 21
71296 Heimsheim/Germany
www.rapidshape.de